

Game Ready™ Protocol

*Instructions for preparing **Game Ready** system for use*

- Load the **Game Ready** with large ice cubes or blocks.
- Fill the machine halfway with cold water.
- Close lid and place unit inside gray insulated bag with the displays facing the zip down flap.
- Insert the red end of the connector hose into the top opening of the Game Ready unit (**Port A**). A distinct “click” will confirm that the hose is connected properly.
- Insert the power cord into the bottom hole on the side of Game Ready (**Port B**).



← **Port (A)**

← **Port (B)**

Operating instructions for **Game Ready™** system

- Press the **POWER** button (1) and wait for the system to set up. You will hear a single beep.
- Press the + button (2) to move the time up to 30-60 minutes.
- Put the temperature knob (4) towards the three snow flakes (coldest).
- Press the **START/PAUSE** button (3) to begin therapy session.



- After 2 minutes, push the temperature control knob (4) halfway between the 3 snowflakes and the 1 snowflake.
- Operate **Game Ready** between 40° F and 50° F for 30 and 60 minutes as recommended (see button 2 shown above).
- When the programmed time period has finished, the **Game Ready** will beep, ending cold therapy and compression.
- Wait one hour (60 min.) and repeat cycle.

**If you have any questions please call
Motion Medical Technology at
(888) 847-6903
For assistance!**